

SpecClean™

PRE-TREATMENT GEL SPRAY

Natural Enzyme Action



Pre-Treatment Gel Spray

Spec Clean™ Pre-treatment is a ready-to-use neutral pH humectant gel pre-treatment spray for instruments and devices in healthcare settings. The gel formula with naturally generated enzyme action prevents bioburden from drying on surfaces. This makes subsequent cleaning more effective and efficient. It is neutral pH and compatible with most materials including stainless steel, aluminum, plastics and glass. This product contains microorganisms which bio-augment cleaning with naturally generated enzymes.

Intended Use

Spec Clean Pre-Treatment Gel is intended for application to soiled instrumentation in healthcare settings at point of use immediately after procedures. Point of use treatment areas include surgical suites, labor / delivery, emergency room, off-site transport and dental offices. This product is also useful for spot treatment in Sterile Processing.

Starts the Cleaning Process With a New and Unique Chemical System

- Naturally generated “always fresh” enzyme action.
- Biodegradable surfactants.
- Targets organic surgical soils including sticky fats
- Testing demonstrates fast, sustained soil breakdown.



Helping you prioritize the safety and well-being of your animal patients during surgical procedures.

Protects Valuable Instrumentation:

- Neutral pH, non-corrosive formula.
- Prevents drying of corrosive blood soils.
- Validated wetting action up to 72 hours.
- Tested for compatibility with most materials (including aluminum).

Improves Productivity at Point of Use and in Sterile Processing

- No sticky residue on carts and trays.
- Rinses off easily.
- Replaces messy liquid soak pans.
- Increased spray area makes full coverage easier to achieve (aerosol).
- Green tint for visible compliance.

Product Capabilities

Clinical and lab testing using blood surrogates, objective soil tests and standardized surface materials demonstrate:

1. Spec Clean Pre-Treatment Gel dissolves many types of target soils including blood, protein and fats starting within 5 minutes of application.
2. It maintains moisture and does not dry hard on surfaces up to 72 hours.
3. It is non-corrosive and protects instrumentation during prolonged holding times up to 72 hours
4. Spec Clean Pre-Treatment Gel is compatible with cleaning processes in automated equipment.

Use Precautions

Spec Clean Pre-Treatment is formulated with naturally generated enzyme action. Follow professional standards and device instructions for ophthalmic instruments and devices.

Spec Clean Pre-Treatment has no special warnings for hazardous chemicals. It is recommended to use Personal Protective Equipment for protection from both chemical and biohazard irritants. At a minimum, eyewear, mask and non-latex gloves are recommended while using this product.

Availability

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com

Product Specifications

Dilution Ratio	Ready-to-Use
pH	Neutral, 7.0 to 8.0
Odor	Non-specific, no fragrance added
Color	Green
Shelf life	2 years from date of manufacture



Spec Clean Pre-treatment is a Ready-to-Use Gel Spray to start soil breakdown, maintain moisture and protect instrumentation during transport.

Package Sizes

Size	CaseQty	Order #
200 mL Spray Can	12	VPREZG/A200-1
32 oz. Spray Bottle	15	VPREZG/32-1

Rev. VPREZG/TDS 23326

© 2023 MicroCare, LLC. All Rights Reserved. "MicroCare Medical", "Spec Clean" are trademarks or registered trademarks of MicroCare, LLC. The information set forth herein is based on data believed to be reliable. MicroCare Medical makes no warranties express or implied as to its accuracy and assumes no liability arising out of its use by others. This publication is not to be taken as a license to operate under, nor infringe upon, any patents not herein expressly described.



6120 East 58th Avenue, Commerce City, CO 80022
(800) 843-3343 • www.microcare.com