

# SpecClean™

EVACUATION SYSTEM CLEANER

Evacuation System Cleaner



## Spec Clean Evacuation System Cleaner

Powerful, non-foaming, concentrated biodegradable evacuation system cleaner that leaves a fresh mint scent.

### Intended Use

Spec Clean Evacuation System Cleaner may be used to clean and deodorize the entire evacuation system, including the high-volume evacuator (HVE), saliva ejector and cuspidor.

### Effective, Yet Gentle:

- Regular use keeps tubing, piping and pumps free-flowing.
- Low pH dissolves mineral soils ensuring complete cleaning, yet is gentle to the evacuation system.
- Cleans and deodorizes the entire evacuation system.

### Safe for All Evacuation Systems:

- Tested on metals and plastics commonly found in evacuation systems.

### Quick Action:

- Dissolves solid debris quickly.
- Rapid removal of organic tissue, fluoride gel, coagulated blood and impression material.



### Applications:

- Clean and deodorize the entire evacuation system.

## Product Capabilities:

Spec Clean Evacuation System Cleaner is a multi-action formula that cleans and deodorizes the entire evacuation system. The highly effective, non-foaming detergent action reduces offensive odors. The superior surfactant leaves no film or residue. Handy unit dose size makes mixing to correct dilution even easier. One ounce (one unit dose packet) diluted with one quart water treats one dental unit (2 - 3 evacuation lines). Use four ounces (four unit dose packets) per gallon of water.

Regular use of Spec Clean Evacuation System Cleaner protects and extends the life of the entire evacuation system and reduces the possibility of system failure. It keeps tubing, piping and pumps free flowing. Spec Clean Evacuation System Cleaner's low pH ensures complete penetration through soil, yet is gentle to the evacuation system and its components.

## Use Precautions

Harmful if swallowed. Do not get in eyes, on skin, or on clothing. Do not store product on counters. Spills may discolor countertops.

## Availability

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit [MicroCare.com](http://MicroCare.com)

## Product Specifications

Active Ingredients	Phosphoric Acid CAS#7664-38-2 Glycolic Acid CAS#79-14-1
pH	Less than 1.5
Odor	Mint
Color	Pink
Shelf life	3 years from date of manufacture

## Package Sizes

Size	Qty	Order #
1 Gallon	4	VPVACNP128-1

