


Spec Clean™ Endoscope Sponge

4 EASY STEPS TO MORE EFFICIENT CLEANING

*"Precognizing should take place at the point of use immediately following the procedure."
ANSI/AAMI ST91:2015*

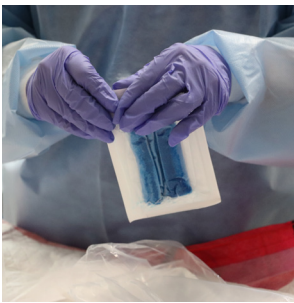


STEP 1

Open and remove pre-saturated sponge and place in water. Squeeze 3-5 times to activate multi-enzymatic detergent.



STEP 2

Lightly squeeze sponge and wipe off exterior surfaces of endoscope.

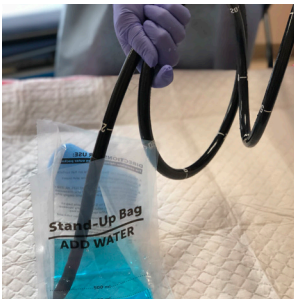

STEP 3

Place distal tip of endoscope into remaining solution for suction (if channel exist).



STEP 4

Endoscope is now ready for transport/transfer.

Spec Clean™ Endoscope Sponge

Individually Vacuum Packed
Order # QEZ2-1



- Reduce time and effort following procedures.
- Endoscope sponge is pre-saturated with detergent for immediate use – No dilution required up to 500ml.
- Powerful multi-enzymatic power starts the cleaning process to prevent dried on soils.



**Scan to
learn more.**