

Spec Clean™ Alkaline Instrument Detergent

Description:

Spec Clean alkaline instrument detergent is an economical, low-foaming, multi-purpose concentrated detergent. Testing confirms a broad range of material compatibility including anodized aluminum, stainless steel and glass or lensed items. It is a non-enzymatic detergent suitable for manual cleaning, ultrasonic instrument cleaning devices, instrument washers and cart washers.

Effective Cleaning Action:

- Objective lab testing verifies effective cleaning action with challenging clinical conditions.
- Formulated for use in a broad range of water conditions with chelating and rinsing agents to ensure best results.
- Low-foaming formula is tested for effective use in automated instrument washers.

Protects Valuable Instrumentation:

- Broad material compatibility includes anodized aluminum, stainless steel, carbon steel and other common materials used in medical instrumentation.
- Non-corrosive formula.
- No added dyes.

Spec Clean alkaline instrument detergent is an economical detergent with effective cleaning agents to help maintain instrument surface quality.



Economical and effective cleaning action.



Product Specifications

Dilution Ratios	1/2 oz. per gallon (4 ml per liter) of warm water for manual, ultrasonic and automated washers and cart washers
pH	8.0 to 10.0
Active Ingredients	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents.
Temperature of Use	In manual applications do not exceed 120°F (49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 150°F (49° to 65°C)
Solution Color	Clear
Odor	Non-specific
Shelf Life	3 year shelf-life, expiration date printed on product

Description	Case Qty	Order#
1 Gallon	4	PCNF128-1

Product Capabilities:

Surfactants and other agents in Spec Clean acts as a chemical team to aid in removing soils during instrument decontamination while protecting surface quality of instrumentation.

Spec Clean is low-foaming and ideal for use in automated washer and cart washer systems. It is also suitable for use when a non-enzymatic detergent is preferred.

Use Precautions:

Spec Clean detergent is a mild irritant. Wear appropriate protective equipment during decontamination procedures to protect against both chemical and biological hazards including eye protection and water-resistant gloves.

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com

MicroCare Medical
6120 East 58th Avenue
Commerce City, CO 80022
(800) 843-3343



Visit Us:
MicroCare.com