

Technical Bulletin

ProLube SCD[™]

Super Concentrated Instrument Lubricant

Description:

ProLube SCD is a lubricant to improve functionality of jointed items. It is designed for use in automated washer/disinfectors and similar equipment. It is super concentrated to reduce package weight and storage space.

Super Concentrated:


- Reduces weight of containers and storage space.
- Economical with reduced cost per cycle.
- Designed for dispensing in automated washer lubrication cycles.

Promotes Best Function of Jointed Items:

- Laboratory testing demonstrates effective lubrication at ultra low dosing.
- Water based lubricant will not create yellow film or become sticky in jointed areas.
- Neutral pH and anti-corrosive components protect delicate instrumentation.

Meets AAMI ST 79 Guidelines for Medical Instrument Lubricants:

- Water soluble with no silicone or mineral oil components.
- Completely permeable to steam sterilization.
- No rinsing required after application and before sterilization.
- Clear formula has no white coloring agents to create potential micro-contamination.

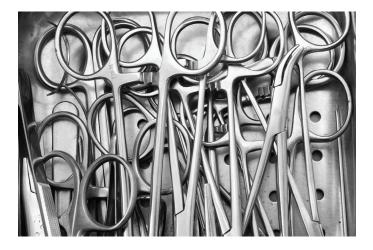

More Power with Less Cost & Weight! 2.5 gallon drums are equivalent to 30 gallon drums of regular dosing detergent.



Extensive lab testing demostrates effective lubricant action

Product Specifications

Dilution Ratio	1/20 to 1/5 oz. per gallon (0.4 mL to 1.6 mL per liter)	
рН	Neutral (6.0 to 9.0)	
Temperature of Use	180°F (82°C) or the temperature set by the washer manufacturer for the lubricant dispensing point or cycle	
Odor	Non-specific, slight alcohol	
Color	Clear and colorless	
Shelf Life	3 years from date of manufacture	
Packaging	2.5 gal. drum, 1 each	

Description	Case Qty	Order#
2.5 Gallon Drum	1	PLCSCD2.5-1

Intended Use:

ProLube SCD Super Concentrated Instrument Lubricant is designed to be dispensed in the lubricant cycle of automated washer/disinfectors and similar equipment for healthcare and other settings. The formula is neutral pH with advanced anti-corrosive components for broad material compatibility with stainless steel, titanium, brass, gold, copper, aluminum, anodized aluminum, heat tolerant plastics and rubber goods. All applications should be according to manufacturer's instructions.

Product Capabilities:

ProLube SCD is a super concentrated lubricant. The neutral pH formula with anti-corrosives improves functionality and extends use life of jointed items used in healthcare and other settings. ProLube SCD is water soluble, completely steam permeable and contains no silicone or mineral oil. It meets AAMI ST79 Guidelines for lubricants intended for use on medical devices. ProLube SCD is clear and has no added coloring agents that could contribute to micro-contamination as the last material applied to instruments prior to sterilization.

Precautions:

Use Certol SCD super concentrates in cleaning equipment designed to dose at more precise volumes from 1/5 oz. per gal. to 1/20 oz. per gal. (0.2% - 0.05%).

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com/Certol.

MicroCare Medical 6120 East 58th Avenue Commerce City, CO 80022 (800) 843-3343



Visit Us: MicroCare.com/Certol

© 2022 MicroCare, LLC. All Rights Reserved. "MicroCare Medical", "ProLube SCD" are trademarks or registered trademarks of MicroCare, LLC. The information set forth herein is based on data believed to be reliable. MicroCare Medical makes no warranties express or implied as to its accuracy and assumes no liability arising out of its use by others. This publication is not to be taken as a license to operate under, nor infringe upon, any patents not herein expressly described.