

Spec Clean™ Pre-Treatment Gel Spray

Description:

Spec Clean™ Pre-Treatment Gel Spray is a ready-to-use, neutral pH humectant gel designed for pre-cleaning soiled surgical instruments and medical devices immediately after use. This gel helps prevent the drying of soils, making subsequent cleaning more effective and efficient. Formulated with biodegradable surfactants and wetting agents, Spec Clean™ Pre-Treatment Gel Spray is compatible with a wide range of materials, including stainless steel, aluminum, plastics, glass, and even delicate ophthalmic instruments.

Improves Productivity at Point of Use and in Sterile Processing:

- No sticky residue left on carts or trays
- Rinses off easily during cleaning
- Replaces the need for messy liquid soak pans
- Bag-on-valve technology provides broader spray coverage with controlled delivery
- Green-tinted gel foam allows visible confirmation of coverage for compliance

Protects Valuable Instrumentation:

- Neutral pH and non-corrosive; safe for extended holding up to 72 hours
- Prevents drying and hardening of blood soils
- Safe for use on sensitive materials, including aluminum and eye instruments
- Made in the USA

Intended Use:

Spec Clean™ Pre-Treatment Gel Spray is intended for use immediately after procedures to pre-treat soiled surgical instruments in healthcare environments. Ideal for:

- Surgical suites
- Labor & delivery rooms
- Emergency departments
- Off-site healthcare facilities
- Dental practices
- Spot treatment in sterile processing departments



Use Spec Clean™ Pre-Treatment Gel Spray to start soil breakdown, maintain moisture and protect instrumentation during transport.



Product Specifications

Dilution Ratio	Ready-to-Use
pH	Neutral, 7.0 to 8.0
Odor	Non-specific, no fragrance added
Color	Green
Shelf Life	2 years from date of manufacture

Description	Case Qty	Order#
12 oz. Spray	12	GEL/12-1

Clinical & Performance Capabilities:

Validated through clinical and laboratory testing with blood surrogates, surface soil tests, and standardized materials:

1. Maintains moisture on contaminated instruments for up to 72 hours
2. Effective on all medical instrument types, including those used in ophthalmology
3. Protects delicate surfaces from corrosion during prolonged hold times
4. Compatible with automated cleaning equipment and manual washing processes
5. Surfactants start breaking down soils immediately upon application
6. Green tint provide visual assurance of full coverage

Use Precautions:

Spec Clean™ Pre-Treatment Gel Spray has no special warnings for hazardous chemicals. It is recommended to use Personal Protective Equipment for protection from both chemical and biohazard irritants. At a minimum, eyewear, mask and non-latex gloves are recommended while using this product. Keep can away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not pierce or burn can, even after use.

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com

MicroCare Medical
6120 East 58th Avenue
Commerce City, CO 80022
(800) 843-3343



Visit Us:
MicroCare.com