

ProE-Vac™ Liquid Evacuation System Cleaner

Description:

ProE-Vac is a powerful, non-foaming, concentrated biodegradable evacuation system cleaner that leaves a fresh mint scent.



Effective, Yet Gentle:

- Regular use keeps tubing, piping and pumps free-flowing.
- Low pH dissolves mineral soils ensuring complete cleaning, yet is gentle to the evacuation system.
- Cleans and deodorizes the entire evacuation system.

Safe for All Evacuation Systems:

- Tested on metals and plastics commonly found in evacuation systems.

Quick Action:

- Dissolves solid debris quickly.
- Rapid removal of organic tissue, fluoride gel, coagulated blood and impression material.

Intended Use:

ProE-Vac may be used to clean and deodorize the entire evacuation system, including the high-volume evacuator (HVE), saliva ejector and cuspidor.

ProE-Vac is a non-foaming, biodegradable evacuation system cleaner that leaves a fresh mint scent. ProE-Vac's multi-action formula cleans and deodorizes the entire evacuation system.



Non-foaming, biodegradable evacuation system cleaner.

Product Specifications

Active Ingredients	Phosphoric Acid CAS#7664-38-2 Glycolic Acid CAS#79-14-1
pH	Less than 1.5.
Odor	Mint.
Color	Pink.
Shelf Life	3 years from date of manufacture.

»» Technical Tip »»



Tidy Tote™ is the easy way to dispense diluted ProE-Vac evacuation line cleaners with OSHA compliant secondary container labeling.

Tidy Tote™ bottle with special adaptor cap to attach dental unit low or high volume suction.

Description	Case Qty	Order#
1 oz. Unit Dose - 24/box	6	PVACU24-1
1 Gallon with Pump	4	PVAC128-1
Tidy Tote - 1 Gallon Empty Dispenser	1	PVACB-1

Product Capabilities:

ProE-Vac is a multi-action formula that cleans and deodorizes the entire evacuation system. The highly effective, non-foaming detergent action reduces offensive odors. The superior surfactant leaves no film or residue. Handy unit dose size makes mixing to correct dilution even easier. One ounce (one unit dose packet) diluted with one quart water treats one dental unit (2 - 3 evacuation lines). Use four ounces (four unit dose packets) per gallon of water.

Regular use of ProE-Vac protects and extends the life of the entire evacuation system and reduces the possibility of system failure. ProE-Vac keeps tubing, piping and pumps free flowing. ProE-Vac's low pH ensures complete penetration through soil, yet is gentle to the evacuation system and its components.

Use Precautions:

Harmful if swallowed. Do not get in eyes, on skin, or on clothing. Do not store product on counters. Spills may discolor countertops.

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com/Certol.

MicroCare Medical
6120 East 58th Avenue
Commerce City, CO 80022
(800) 843-3343



Visit Us:
MicroCare.com/Certol