

ProWash™ Multi-purpose Instrument Detergent and Cart Wash

Description:

ProWash is an neutral pH, low-foaming, multi-purpose concentrated instrument detergent. Testing confirms a broad range of material compatibility including anodized aluminum, stainless steel and glass or lensed items. ProWash is a non-enzymatic detergent suitable for manual cleaning, ultrasonic instrument cleaning devices, instrument washers and cart washers.

Effective Cleaning Action:

- Objective lab testing verifies effective cleaning action with challenging clinical conditions.
- Formulated for use in a broad range of water conditions with chelating and rinsing agents to ensure best results.
- Low-foaming formula is tested for effective use in automated instrument washers.

Protects Valuable Instrumentation:

- Broad material compatibility includes anodized aluminum, stainless steel, carbon steel and other common materials used in medical instrumentation.
- Non-corrosive formula.

ProWash is a premium neutral pH multi-purpose detergent combining a powerful cleaning system with excellent material compatibility.



Robust formula for wide variety of applications.



Product Specifications

Dilution Ratios	1 oz. per gallon water for manual and ultrasonic cleaning. 1/2 oz. per gallon water for automated washers and cart washers depending on soils and water quality
pH	7.0 to 9.0
Active Ingredients	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents
Temperature of Use	In manual applications use 90° to 120°F (32° to 49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 150°F (49° to 65°C)
Solution Color	Blue
Odor	Non-specific
Shelf Life	3 year shelf-life, expiration date printed on product

Description	Case Qty	Order#
1 Gallon	4	PW128-1
5 Gallon Drum	1	PW050-1
15 Gallon Drum	1	PW150-1
30 Gallon Drum	1	PW300-1
55 Gallon Drum	1	PW550-1

Product Capabilities:

Surfactants and other agents in ProWash act as a chemical team to aid in removing soils during instrument decontamination while protecting surface quality of instrumentation.

ProWash is low-foaming and ideal for use in automated washer and cart washer systems. It is also suitable for use when a non-enzymatic detergent is preferred.

ProWash is useful for cleaning ophthalmic instruments and devices when manufacturer's instructions recommend a neutral detergent formulated without enzymes.

Use Precautions:

ProWash detergent is a mild irritant. Wear appropriate protective equipment during decontamination procedures to protect against both chemical and biological hazards including eye protection and water-resistant gloves.

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com/Certol.

MicroCare Medical
6120 East 58th Avenue
Commerce City, CO 80022
(800) 843-3343



Visit Us:
MicroCare.com/Certol