

More Power
with Less Cost & Weight!

2.5 gallon drums are equivalent to 30 gallon drums of regular dosing detergent.

ProWash SCD™ Super Concentrated Instrument Detergent and Cart Wash

Description:

ProWash SCD is a neutral pH detergent used when cleaning instruments and devices in automated equipment. It is super concentrated to reduce package weight and storage space.

Value and performance in super concentrated formula:

- 2.5 gallons of ProWash SCD delivers over 300 loads of clean instruments in a standard instrument washer or about 50 loads in cart washer.*
- 2.5 gallon container replaces 15 to 30 gallon drum.
- Reduced weight and footprint for easier handling and storage.

Standardize with a versatile detergent:

- Effective, economical cleaning in instrument washers AND cart washers.

Convert with confidence!

- Testing validates effective cleaning at ultra low dose points.
- Developed using ASTM D8179 guidelines for medical detergents.
- Neutral pH with superior anti-corrosive system offers broad material compatibility and instrument protection including anodized aluminum containers.

Intended Use:

ProWash SCD Super Concentrated Detergent and Cart Wash is non-foaming and formulated to promote ideal pressure and spin action in automated washer/disinfectors and cart washers. This product is used to clean instruments and appropriate devices used in healthcare and other settings. The formula is neutral pH with advanced anti-corrosive components for broad material compatibility with stainless steel, titanium, brass, gold, copper, aluminum, anodized aluminum, heat tolerant plastics and rubber goods. All applications should be according to manufacturer's instructions.

*Dosing at 1/10 oz./gal, 1 mL/liter or 0.1%.



Premium neutral pH multi-purpose formula



Product Specifications

Dilution Ratios	1/40 to 1/5 oz. per gallon (0.2 mL to 1.6 mL per liter)
pH	Neutral
Temperature of Use	120°F to 150°F (49°C to 66°C)
Odor	Non-specific, fresh
Color	Natural clear yellow
Shelf Life	2 years from date of manufacture
Packaging	2.5 gal. drum, 1 each

Description	Case Qty	Order#
2.5 Gallon Drum	1	PWSCD2.5-1

Product Capabilities:

ProWash SCD contains surfactants and other components to rapidly dissolve a wide range of surgical soils. The neutral pH formula complies with manufacturer's instructions when specifying neutral pH and non-corrosive formulas. It is appropriate for use on heat tolerant instruments and devices indicated for cleaning in automated washing equipment. Engineered formula enhances spin and cleaning action in all models and designs of automated washers and cart washers.

Use Precautions:

Use MicroCare Medical SCD super concentrates in cleaning equipment designed to dose at more precise volumes from 1/5 oz. per gal. to 1/20 oz. per gal. (0.2% – 0.05%).

Availability:

Stocked by MicroCare Medical distributors. Contact MicroCare Medical for a distributor near you or visit MicroCare.com/Certol.

MicroCare Medical
6120 East 58th Avenue
Commerce City, CO 80022
(800) 843-3343



Visit Us:
MicroCare.com/Certol