

NEW **MicroCare PRE-TREATMENT SPRAY** ProEZ Gel NATURALLY GENERATED ENZYME ACTIVITY Faster Applications 72-Hour Action **Free Rinsing CONTENTS: 200 ML** Reorder No. PREZG/A200 Keep Out of Reach of Children Do Not Take Internally



Scan for more info.


ProEZ Gel"

PRE-TREATMENT SPRAY

Improve Instrument Pre-Treatment Processes
Fulfills AAMI and AORN Recommendations

3 EASY STEPS

FOR BETTER, FASTER CLEANING
WITH LESS WASTE

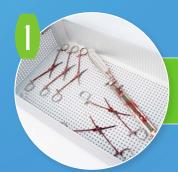

Open soiled instruments and place on tray.



Unlock cap by turning counterclockwise.
Start dispensing by pressing button down with fingertip.



Use sweeping motion to thoroughly wet instruments. Lift finger when done.

Instruments are now ready for transport.